



## Components

1) The Universal Transmitter LT-900 with pre-installed transmitter code.



2) Wrist Strap (not shown)

LT-900 Long Range Transmitter is Long Life battery operated and connects to Sensor Pads. The transmitter is attached to posts on Wheelchair, Beds, etc. with Fast Install Velcro straps. When sensor is activated by weight of patient (getting ON or OFF sensor pad), the transmitter sends the signal to the Enunciator. ON-OFF switch allows caretaker to attend patient. The LT-900 transmitter is selectable to monitor chair, bed, floor and rail/bed sensor pads. Transmitter is selectable for INSTANT or DELAY (2 sec.) operation, allowing patient to move around in chair or bed without sending alarm!



The LT-900 Long Range Transmitter has a switch to turn the transmitter on for instant or delayed operation or off to allow the caregiver to work with a patient and not have the alarm sound. The led will flash when transmitter is activated. In an alarm condition make sure both transmitter and receiver units are reset to there non-alarm condition when situation has returned to normal.

There is also a switch to set the sensor for Normally Open, position O, to activate the alarm when weight is getting on the pad or Normally Closed, position C, to activate the sensor when weight is getting off the pad.



We strongly recommend that you test the transmitter with the receiver/enunciator system frequently. From the time of installation, it is absolutely necessary to test the system at least once a week to ensure that the LT-900 (transmitter) will be ready when needed.

### Changing Batteries:



The batteries should be replaced every year or as needed with your transmitting unit. To replace the batteries in your transmitter you will need one 3-volt Lithium battery (CR123A or equivalent). Carefully remove the screws on the transmitter case and separate the transmitter housing.

Remove the old battery from the battery holder and place the new battery with the positive end of the battery facing up towards the bottom of the unit towards the RJ22 connector. **BE SURE TO PRESS THE RESET BUTTON AFTER INSTALLING THE NEW BATTERY.**

## Universal Transmitter

### LT-900

## USER MANUAL QUICK START GUIDE

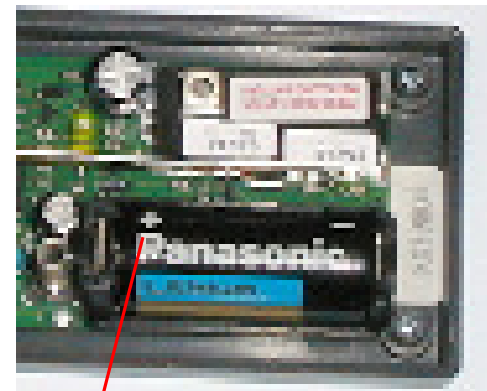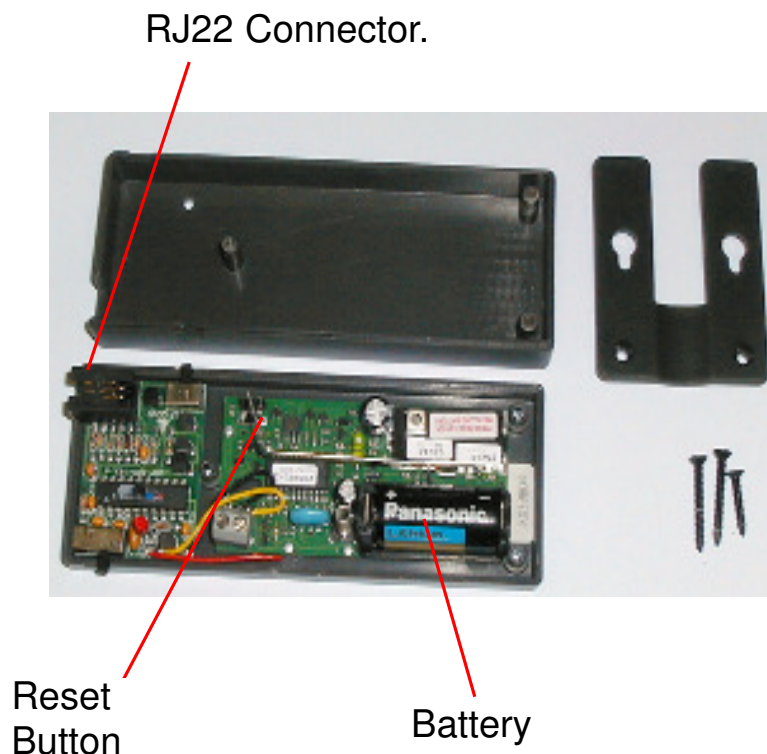

+ side

Then reassemble the main transmitter case. (DO NOT OVER TIGHTEN SCREWS) After changing the batteries, be sure to test the transmitter to make sure you have installed the batteries properly.

Now you can feel Safe, Secure and at Ease whenever patients, family members, or friends are confined to bed, home or facility and need fast assistance.

