


P- 3- 22 Bed Sensor Pad 14" x 24" Mounts on Bed Mattress, bathtub, toilet, etc.

USER MANUAL QUICK START GUIDE

Components:

1. Sensor Pad

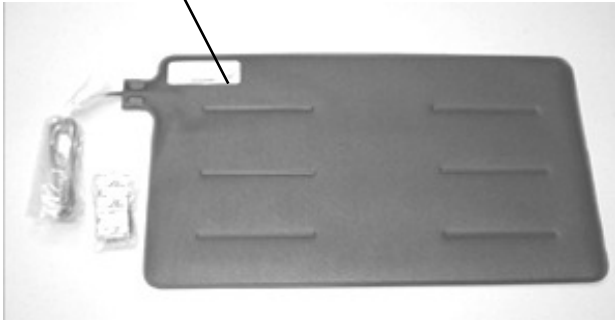

Step 1.

Uncoil the cord from the sensor pad. Place the sensor pad on the bed with the cord going down under the mattress and towards the back of the bed. Place the pad at waist level as shown. For high risk patients place the pad at a point across the patients shoulder blades.

The RJ22 connection at the end of the cord may be connected into any of Emergency Caller Products alarm units such as the Patient Alert, the Extended Range Transmitter, TR-300, or the Long Range Transmitter, the LT-900. Consult your Emergency Caller Products sales representative to learn how these other products work with this model sensor pad. The pad may also be connected into other products that will receive the RJ22 connector.

Sensor Pads detect both Applied Weight "Patient ON" pad or Removed Weight "Patient Off". Selection made by switch on Universal Transmitter Unit or other monitoring products. Can be used to detect patient in bathtub or shower for entry or exit (weight off or weight on). Sensor pads are slip resistant and completely waterproof. Complete with velcro tabs and 8 ft. leads (except Chair-seat Pad with a 4 ft. lead). Lifetime sensor pads replace all disposable sensor pads. Emergency Caller Sensor pads have been used for over 25 years in security applications. They are durable, safe and withstand both outdoor and indoor environments. With lifetime guarantee you never again need to be concerned about constant replacement and expensive costs.