

## XP-11-900


### Overview

The XP-900 series Transmitter Strips and Pads are effective monitoring devices when used with compatible 900MHz receivers.

When weight is applied to the sensor strip or sensor pad, a signal is sent to the receiver. Depending of the model chosen, receivers can notify caregivers with flashing strobe and audio alarm, or by dialing phone numbers and delivering a voice message, or with audio alarm and displayed information such as patient name, event type, and location.

### Strip or Pad Transmitter Preparation

XP-11-900 is shipped with a thin piece of plastic insulation between the battery and its holder to prevent possible battery discharge during transit. To activate the battery, follow the procedure below. Disassembly/reassembly of the transmitter module also applies to future battery replacement.

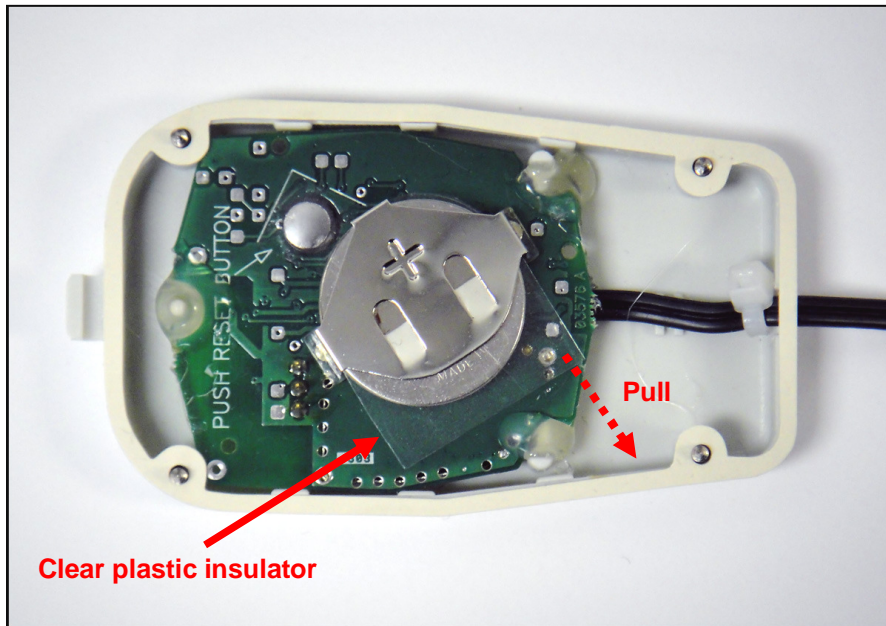

1. Remove the four small phillips screws located on the bottom of the transmitter module.
2. Carefully separate and lift off the top of the module housing leaving the gasket in place.

3. Pull out the clear plastic insulator which will enable the battery to make contact and power the Transmitter.
4. Carefully match the top housing to the bottom housing, making sure that the gasket is properly placed.
5. Re-install the four phillips screws.

Note: Battery requirement is CR2450 3V Lithium Coin

### Transmitter and Receiver

When Receivers and Transmitter Strips or Pads are purchased together, the two devices are factory paired to work as a system. In this case, you will only need to follow the instructions supplied with the Receiver, and then test the system by applying pressure to the Strip or Pad.

If a 900MHz Receiver has been purchased separately, please contact our technical services department for transmitter products to receiver pairing instructions.

### Range

Because 900MHz range is affected by conditions such as physical obstructions and the types of material they are made of, there is no exact distance measurement for effective signal transmission between transmitters and receivers.

Our wireless products incorporate the most advanced 900MHz technology available. Range typically exceeds most users' requirements.

*Thank you for choosing...*



**HealthCare Security Products**  
**Emergency Caller Products** GETS HELP FAST

**United Security Products, Inc.**

(T) 800-227-1592

(F) 858-413-0124

Email: [sales@unitedsecurity.com](mailto:sales@unitedsecurity.com)

[www.unitedsecurity.com](http://www.unitedsecurity.com)

[www.emergencycaller.com](http://www.emergencycaller.com)