

# Patient Alert PA- 3 BED Fall Prevention System QUICK START GUIDE

Emergency Caller Products GETS HELP


Emergency Caller Products (T) 800-227-2474 (F) 800-762-7613  
www.emergencycaller.com sales@emergencycaller.com  
U.S.A. and World Patents Issued and Pending on Certain Products



Follow these simple instructions to set up your system


## Step 1. Positioning the Pad.

Place the sensor pad on the Bed - underneath mattress cover or bottom sheet with the cord placed unobtrusive and routed to Patient Alert unit.

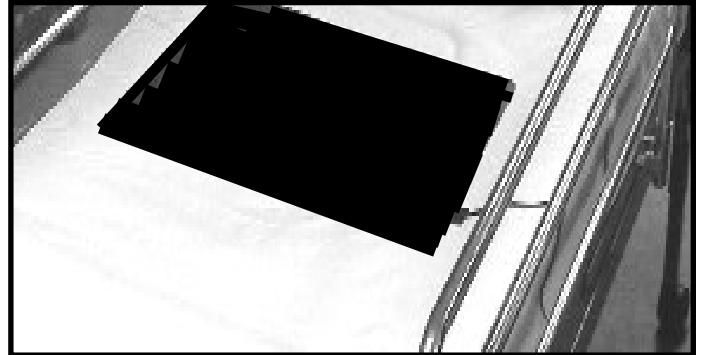

## Step 2. Connect the cord.

Place the connector plug on the end of the pad into the socket on top of the Patient Alert unit as shown.

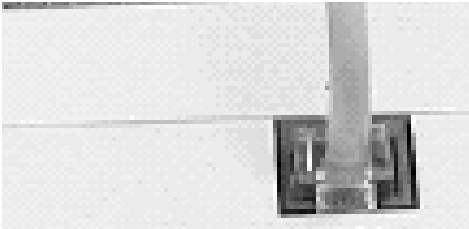

## Step 3. Position the Alert Unit.

With the Patient Alert still in the off position, attach the Alert unit to desired location on bed using the Mesh Bag provided. Patient Alert sound is designed to alert caretaker while alerting patient to return to bed.

## Step 4. Operate the system.

Place the patient on the bed. Now you are ready to activate the Patient Alert unit.

### A- Momentary Activation ( MOM)

Switch Alert unit to **MOM** position. This will activate Alarm when patient attempts exit. The Patient Alert unit will Automatically reset and silence alarm when patient returns to Sensor Pad position or unit is turned off by caretaker.

### B- Continuous Activation ( CONT)

When the Alert switch is placed in **CONT** position and patient exits Sensor Pad, Patient Alert will sound alarm continuously until turned off by caretaker



**\*IMPORTANT**  
**If Patient Alert System Fails -  
Replace Battery.  
Replace Battery Periodically**  
**Test System Before Each Use**

**PLEASE SEE REVERSE SIDE FOR  
LIFETIME SENSOR PAD  
INSTALLATION INSTRUCTIONS**

# LIFETIME SENSOR PAD MOUNTING INSTRUCTIONS

## PLEASE READ CAREFULLY

### WHEELCHAIR OR CHAIR INSTALLATION



Mount Sensor Pad ACROSS Wheel Chair Seat with wire leads routed toward back to Patient Alert, sensor units or transmitter.


### BED INSTALLATION




Mount Sensor Pad ACROSS Bed with wire leads routed toward back to sensor unit.  
Mount at patient HIP position on bed for normal response time use.  
Mount at beneath shoulders position for Quick Response. Sensor Pad may be placed beneath bed cover or bottom sheet.

**DO NOT! MOUNT SENSOR PADS AS SHOWN BELOW !**

**DAMAGE WILL OCCUR TO SENSOR PADS AND VOID LIFETIME WARRANTY.**

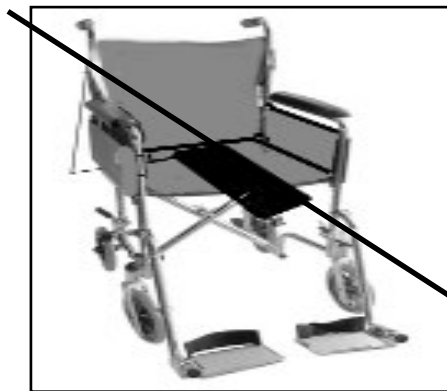

**DO NOT MOUNT SENSOR PAD LENGTHWISE ON WHEELCHAIR OR BED. THIS WILL BEND SENSORS INSIDE PAD AND **VOID LIFETIME WARRANTY****

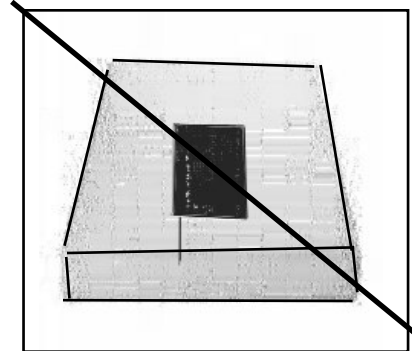

**DO NOT FOLD, BEND OR ROLL SENSOR PAD**

**\* ( This will Void Lifetime Warranty)**

### **IMPORTANT WARRANTY INFORMATION**

**Do Not Return Defective Sensor Pads to Distributor!**

**If you believe Sensor Pad is defective, Call 1-800-227-2474**

**To speak with Customer Service Tech and receive Return Authorization Number**

**Emergency Caller Products** 